Installation for Macbook


NightWatch Update (v14.3)

This document gives step by step instructions for installing the 'NightWatch Updater' application on your **Macbook**. You can use this application to update your NightWatch so that you can use the new Portal.

Step 1


Download the 'NightWatchUpdater-20210126_v143.dmg' file. You can download this file at www.nightwatchepilepsy.com/update. Click the 'Mac' button under the 'NightWatch update software' banner to start the download.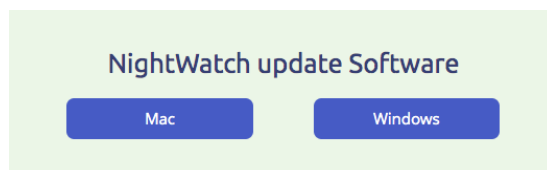

1. Download the file via nightwatchepilepsy.com/update

Step 2

Find the downloaded 'NightWatchUpdater-20210126_v143.dmg' file on your computer (click your 'Downloads' folder). Double click this file. The application will be installed and you can then open it. Click the little application icon as pictured here on the right.

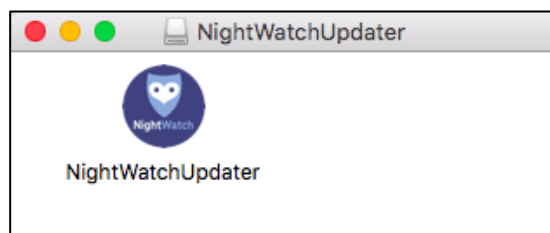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
3. Click 'OK' for this security notification

Step 4

You now need to confirm that you want to open this application via your 'System preferences'. Click on the 'Apple' icon at the top left of your screen and select 'System preferences...'.
(Note: The original text contains a typo: "via your 'System preferences'". It should be "via your 'System preferences'".)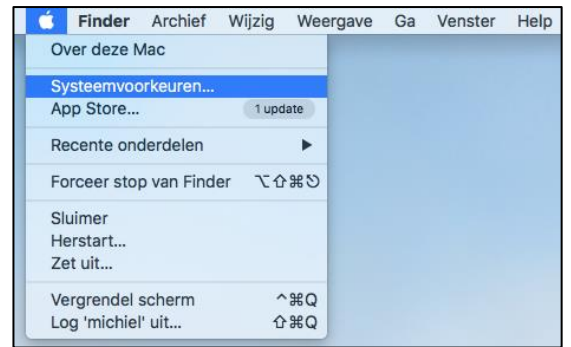

4. Click on the little Apple at the top left and then click 'System preferences...'

Step 5


Open 'Security and privacy' (or click 'Back' if another page opens).

5. Open 'Security and privacy'

Step 6

Then click 'Open anyway'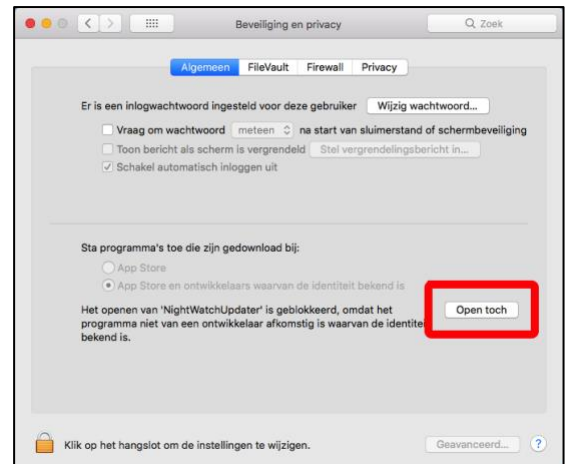


6. Click 'Open anyway'

Step 7

The 'NightWatch Updater' application has now been installed and opened. You can use this application to find and update your NightWatch.

Click 'Info' in the application for more information about this.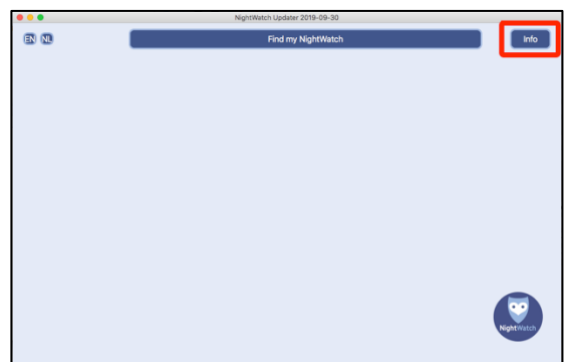

7. The application is open. Click 'Info' for more information.