



INFO

This document briefly describes which changes have been released in the v12.7 version of the NightWatch since the v11.6 version. Seizure detection algorithms have not been changed.

CHANGES

1. NEW FEATURE - When connected to the internet, the NightWatch base station is able to send an email containing data of last night. It sends data from start-recording till stop-recording.
2. NEW PORTAL - Users can access their data via a first version of the NightWatch Portal.
3. NEW FEATURE – created a possibility to toggle relay settings for technical alarms.
4. NEW FEATURE – created a possibility to send alarms over an IP server (API).
5. BUG FIX – The arm module indicated with a green LED that it was charging while being switched off. Though, the arm module does not charge when switched off. The arm module now indicates that it is not charging when switched off while on the charger.