



## INFO

---

This document briefly describes which changes have been released in the v14.1 version of the NightWatch since the v13.2 version. Seizure detection algorithms have not been changed.

## CHANGES

---

1. NEW DEFAULT SETTING – The default setting of the Relay switch changed. The most important technical notifications (Bad-Signal, No-Signal, Connection-Lost, Battery-Low) will trigger the relay switch by default now.
2. NEW API FEATURE – Added two API messages that will be send over the second IP server. First is a 'All-Good' message, indicating that a heart rate has been detected. Second is a 'In-Rest' message, indicating that the person is horizontal and lies still for at least 2 minutes.
3. NEW API FEATURE – The 'Keep-alive' message will now be sent every ten minutes, even when another message has been sent in the meanwhile.